

# CARE AND CLEANING GUIDE



## CARE AND CLEANING GUIDE FOR OUR COVER FABRICS.

Thank you for choosing a GREINER brand product – congratulations to a top-class choice – particularly when it comes to upholstery and fabrics.

Each type of cover fabric promises to stand for long-lasting quality, if it is disinfected and cleaned in a regularly and durable manner. Quick-and-easy hygiene measures help to fulfill the daily care plan.

This manual will inform you about all aspects of use and application of suited cleaning and disinfection products.

Oil, fats, ink, blood, urine and organic acids must be cleaned off immediately using a damp microfibre cloth and either lukewarm water or a mild washing-up liquid in warm water. Do not use any solvents, chlorides, polish (including wax polish) or chemical cleaning agents. Discoloration due to denim jeans or other clothing is not covered by any manufacturer warranty.



# Tuxor.

# compact.

### Recommended disinfectants by the fabric manufacturer

Adexano	Bacoban WB 1%	✓
Antiseptica	Acrylan	✓
B. Braun Melsungen AG	Meliseptol Foam / Hexaquart plus lemon fresh / Hexaquart forte 2%	✓
Bode Chemie GmbH	Dismozon Pur / Microbac Forte / Microbac Tissues / Bacolloicid Spezial / Kohrsolin FF	✓
CosiMed GmbH	CosiMed 7,5%	✓
Dr. Schumacher	Optisept 7% / Cleansept Wipes / Cleansept 2% / Biguanid Fläche	✓
Dürr Dental GmbH & Co. KG	FD 360 undiluted	✓
Ecolab	Sani-cloth active / Incidin rapid	✓
Franken Chemie GmbH Co KG	Franko-cid N0, 25% / Dessan 2 0, 25%	✓
Metasys	Green & clean SK start	✓
Lysoform Dr. Hans Rosemann GmbH	Lysoformin spezial 0	✓
Schülke & Mayr GmbH	Microzid sensitive Liquid / Mikrozid universal Liquid / Perform	✓
Pino	Septapin Aquaquick alkoholfrei	✓

### Recommended disinfectants by the fabric manufacturer

Akzo Nobel	Natriumhypochlorid	✓
Bode Chemie GmbH	Bacillo AF / Bacillo 30 Sensitive Foam + Tissues / Cutasept / Mikrobac forte	✓
Braun	Meliseptol Foam Pure / Softasept N / Meliseptol / Helipur / Melsitt / Melsept	✓
Desomed	Desomed Rapid	✓
Dr. Schumacher	Descosept AF	✓
Ecolab	Incidin foam / Incidin Plus / Incidin perfekt / Incidin liquid	✓
Favodent Karl Huber	Flora quick	✓
Henkel	Incidur-Spray	✓
Lysoform	Fugaten Spray	✓
Orochemie	B45	✓
Peppler	Micro clean / Pepp clean	✓
Reckitt Benckiser	Sagrotan med	✓
Ruck	Sprühdesinfektion	✓
Schülke & Mayr	Kodan Tinktur Forte / Esemfix / Octenisept / Mikrozid AF	✓
Unident	Unisepta foam	✓

# electra.



Recommended disinfectants by the fabric manufacturer		
Adexano	Bacoban WB 1%	✓
Antiseptica	Acrylan	✓
B. Braun Melsungen AG	Meliseptol Foam / Hexaquart plus lemon fresh / Hexaquart forte 2%	✓
Bode Chemie GmbH	Dismozon Pur / Microbac Forte / Microbac Tissial / Bacolloicid Spezial / Kohrsolin FF	✓
CosiMed GmbH	CosiMed 7,5%	✓
Dr. Schumacher	Optisept 7% / Cleansept Wipes / Cleansept 2% / Biguanid Fläche	✓
Dürr Dental GmbH & Co. KG	FD 360 undiluted	✓
Ecolab	Sani-cloth active / Incidin rapid	✓
Franken Chemie GmbH Co KG	Franko-cid N0, 25% / Dessan 2 0, 25%	✓
Metasys	Green & clean SK start	✓
Lysoform Dr. Hans Rosemann GmbH	Lysoformin spezial 0	✓
Schülke & Mayr GmbH	Microzid sensitive Liquid / Mikrozid universal Liquid / Perform	✓
Pino	Septapin Aquaquick alkoholfreie	✓

The highest recommended concentration was tested in 50 applications each. The instructions for use of the detergents must be observed in every case. An impairment of the fabric in long-term use is not excluded.

The change of cleaners and disinfectants should be avoided especially when using over a longer period, otherwise a degradation to the materials' properties can not be ruled out. Long-term use of alcohol-based cleaners or other solvents may negatively affect the coated surface and promotes embrittlement of the cover.

As we have no control over the ingredients and manufacture of the recommended disinfectants, we have to reject any guarantee on the disinfectant resistance of the GREINER product.

Lacquers, strong cleaners, detergents, xylene-based solutions, acetone, or MEK cause immediate damage and contribute to the deterioration of the material. The use of such cleaners is at the owner's risk. In order to check a not specified disinfectant on how well its impact is to the fabric a test at an invisible place is recommended.

GREINER will not assume responsibility for discolouration caused by external contaminants and possible permanent staining caused by this phenomenon.



## CARISMA

**The best care right from the start.**

Besides disinfection and cleaning of our products, proper care is particularly important to maintain their longevity and aesthetic appeal. For this purpose, we recommend the use of our **CARISMA cleaning & care lotion** as well as our **special care set** with the reusable **high-pile cloth**. In this way, you not only support the value retention of your products, but also ensure a pleasant and hygienic environment.

**The result:** your upholstery retains shine and glow despite heavy use.



GREINER GmbH  
Wettestraße 1 · 74385 Pleidelsheim · Germany  
T +49 7144 81120 · F +49 7144 8112 99  
info@greiner-gmbh.de · www.greiner-gmbh.de